




## Advanced Healing Hydrocolloid Dressing (Plus)

### Description

This Hydrocolloid Dressing contains carboxy-methylcellulose (CMC) which absorbs wound fluid and forms a gel to create a moist wound environment for the wound to heal. It is a top film that is bacteria-proof and waterproof.

### Application of dressing

1. Wash hands before applying the dressing.
2. Clean and dry the wound and surrounding skin gently. Ensure no cream, ointment or similar product is on the skin.
3. Remove dressing from pouch.
4. Apply the dressing so that the sticky side sits directly over the wound.

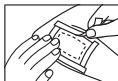

When the product absorbs wound exudate, a white gel is formed and the appearance of the product will become white. The product should be changed when clinically indicated, when visible signs of exudate approach the edge of the product or after 7 days.

### Removing the dressing:

5. Gently lift the corners of the product and remove it from the wound. Soaking the dressing with saline water where the wound is in contact can assist with removal.

### Precautions for use

Seek medical advice immediately for all serious wounds and burns or if you experience redness or discomfort near the wound. A health care professional should frequently inspect and manage infected wounds, diabetic wounds and wounds which are solely or partially caused by arterial insufficiency, in accordance with local guidelines.

Dressing is sterile unless the individual pouch is opened or damaged. Do not use the dressing if the pouch is open or damaged. Single-use. Reuse of this single use product may create a potential risk to the user. Store in a cool, dry place (<35 degrees Celsius).

**Manufacturer:**  
BPositive Pte Ltd

[www.bpositivehealthcare.com](http://www.bpositivehealthcare.com)

### Intended Use

The intended purpose of our dressing is to provide an optimal condition for wound healing and create a barrier to protect wounds from bacterial and pathogens. Can be used in the management of low to moderate exuding wounds like venous leg ulcers, pressure ulcers, superficial burns, superficial partial - thickness burns, donor sites, postoperative wounds, skin abrasions and small wounds. It can also be used to rehydrate necrotic tissue, which is then removed by autolysis

### Importer:

**BPositive Pte Ltd**  
Singapore Business Federation Centre  
160 Robinson Road, #14-04,  
068914 Singapore

### Do not use

Re-use of the single use product may create a potential harm to the user. Reprocessing, washing, disinfection and/or sterilization may compromise product characteristics, causing additional risk of physical harm or infection to the user.

### BPositive Healthcare Pty Ltd

Suite 18, 11 Wilson Street, South Yarra VIC,  
3141 Australia

Hydrocolloid dressings are generally contraindicated wounds with heavy exudate, tunneling wounds or sinus tracts, infected wounds, wounds with exposed tendon or bone, or wounds with fragile periwound skin.

### Local Authorised Representative:

**T T MEDICAL MANAGEMENT SDN. BHD.**  
A-3A-03 EkoCheras, No:693, Batu 5 Jalan  
Cheras, 56000 Kuala Lumpur, Malaysia  
TL : +603 9134 5391

bpositive accepts no liability for any injury or loss that may arise if this product is used in a manner contrary to bpositive's current recommendations.

Basic UDI-DI 888501891021910BAV3

